

## Installation and Use

### Monitor Corner Guards


STERIS Monitor Corner Guards help avoid accidental injury from contact with monitor corners.

Your kit includes four (4) corner guards, two (2) short length straps, and two (2) long length straps. (Note: For some monitor sizes, all four straps are the same length).



### Usage Precautions

- To avoid damage to monitors and ensure proper fit of corner guards used with swing arm mounted monitors, never use corner guard straps as a handle. Move monitors by handles or monitor casing only.

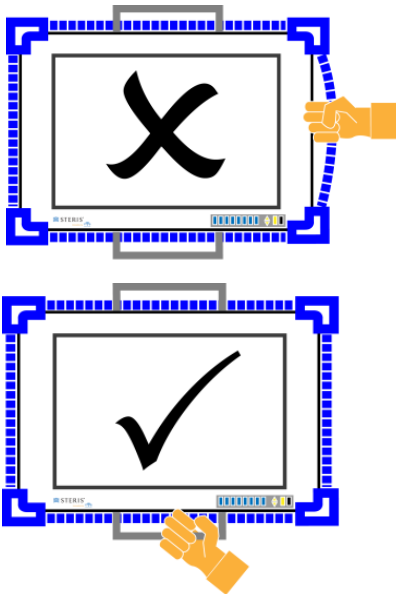

- Ensure that corner guards are securely installed before using the monitor.
- Make adjustments when the monitor is not in use.
- Should it become necessary to make an adjustment during a surgical procedure, first move the monitor away from the surgical site.

### Maintenance

- Disinfect with the medical grade disinfectant solution recommended by the monitor manufacturer for disinfecting the monitor. Refer to your monitor user manual for cleaning solution recommendations.
- Maximum temperature for materials (straps and corner cushions): 230° F /110° C.
- Hospital should determine frequency of replacement based on cleaning, disinfecting, and sterilization policy.

### Specifications

- STERIS Corner Guard Models are for use with the following STERIS Surgical Grade Monitor Models:  
MON-STE19HD, MON-STE21HD, MON-STE21HD(M),  
MON-STE24HD, MON-STE24HD(M), MON-STE26HD2,  
and MON-STE26HD2M.
- Strap Lengths:  
For 19" Monitors (P/N: ACC-0001): 10 1/4" (2), 11 3/4" (2)  
For 21" Monitors (P/N: ACC-0002): 11 3/4" (4)  
For 24" and 26" Monitors (P/N: ACC-0003): 11 3/4" (2), 16" (2)
- Corner Guards:  
Outside Dimensions: 1.75" W x 2.1" L x 2.13" H  
Inside Dimensions: 1.58" W x 1.86" L x 1.86" H
- Material Certifications:  
Elastocil® LR3003/20/30/40/50/60/70 A,B

*See reverse for installation instructions.*

# Installation and Use

## Monitor Corner Guards




### Important Safety Instructions

- Read these instructions.
- Keep these instructions.
- Follow all instructions.
- **CAUTION!** Ensure that the monitor is powered off before attaching STERIS Monitor Corner Guards.
- To ensure proper fit, use only the Monitor Corner Guard set intended for use with your monitor model.

### Installing STERIS Monitor Guards


1. On a flat surface, lay out the corner guards and straps as shown in the photo below. When all straps are the same length, lay out the straps to form a square instead of a rectangle.

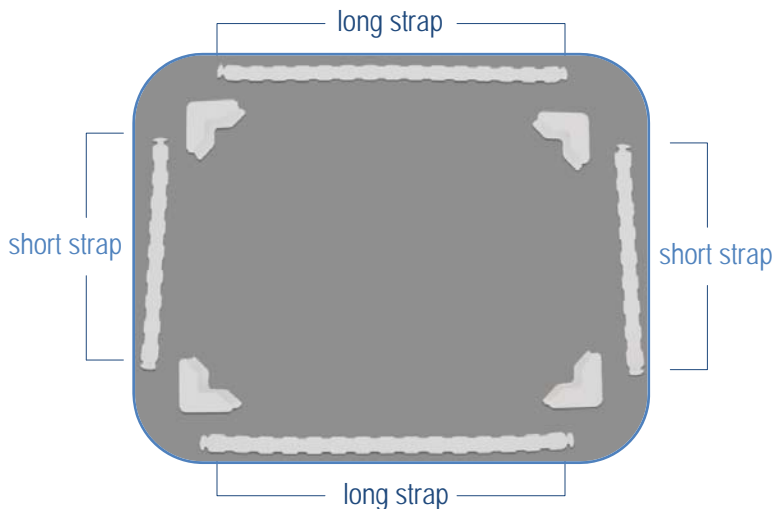

2. On either end of each strap is a fastener. Thread each fastener through the nearest corner guard slot, pressing the fastener through the slot from the interior to the exterior, as shown below. Ensure that the straps are not twisted.



3. The fully assembled corner cushion assembly is shown below.

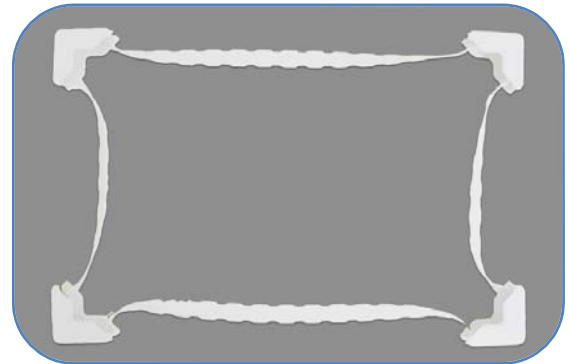

4. Pick up the entire assembly by the top two corner guards. Seat the top guards snugly onto the top monitor corners, as shown in the photo below.



Keep the rest of the strap assembly in the front of the monitor to avoid entanglement with cables attached at the rear.

5. Ensure that the strap stretched across the top of the monitor is straight (untwisted).
6. Seat the bottom guards into position, ensuring that all straps are straight (untwisted), as shown below.



*See reverse for usage precautions, maintenance, and specifications.*