

Installation and Use

Monitor X-Ray Clips

STERIS Monitor X-Ray Clips hold X-Ray film to STERIS 42" monitors (model MON-STE42-MD).

This monitor includes a light box feature that can be used for viewing medical films such as X-ray, CT scan, mammography, Magnetic Resonance Imaging (MRI), PET scan, ultrasound, etc. The light box feature is for reference use only. Do not use for diagnosis.

Your order includes two (2) Monitor X-Ray Clips.



Important Safety Instructions

- Read these instructions.
- Keep these instructions.
- Heed all warnings.
- Follow all instructions.
- **CAUTION!** Ensure that the monitor is powered off before attaching STERIS Monitor X-Ray Clips.
- **CAUTION!** To avoid damaging plastic thumbscrews, hand tighten only. Do NOT use pliers or other tools.
- STERIS Monitor X-Ray Clips are for use only with STERIS monitor model MON-STE42-MD. Do not use with other monitor makes or models.

Maintenance

Disinfect with the medical grade disinfectant solution recommended by the monitor manufacturer for disinfecting the monitor. Refer to your monitor user manual for cleaning solution recommendations.



Attaching STERIS Monitor X-Ray Clips

1. Unthread the thumbscrew of the first clip until there is enough clearance to lay the clip flat over the top edge of the monitor, as shown below.



2. Tighten the thumbscrew into the back of the monitor until the clip is attached firmly. To avoid scratching the monitor glass or paint, take care not to over tighten.

Adjust the tightness of the thumbscrew until the proper tension is achieved. When the clip is attached properly, the clip head will rest lightly against the monitor glass. As with a regular light box clip, you will be able to slip an X-Ray film underneath and the film will remain firmly in position.

3. Repeat for the second clip, positioning it about six inches (6") from the first, as shown below.

